# Sprühdesinfektion AF *aldehydfrei APPLE*

(GB) **Spray disinfectant AF** VAH/DGHM-  
*aldehyde-free APPLE* gelistet

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## **GB** Spray disinfectant AF *aldehyde-free APPLE*

Contents: e 10 ltr. UN 1987 REF 100 711

Highly-effective, ready-to-use spray disinfectant agent for the fast disinfection of small areas, handles and angled fittings in the dental medical area, with apple scent.

**Type of application: According to MPG:** ready-to-use quick disinfection agent for medical and dental use for the disinfection of non-invasive medical devices. **According to the Biocide Directive:** ready-to-use quick disinfection agent for fast disinfection of small surfaces.

**Application:** Spray on items to be disinfected from a distance of approx. 30 cm until completely covered. Not suitable for use on acrylic glass.

**Acting range:** Bactericidal (incl. MRSA) acc. to VAH/DGHH, EN 1040, EN 1276; levurozid (Candida albicans) acc. to VAH/DGHH, EN 1275, EN 1650; tuberculocidal acc. to VAH/DGHH, EN 14348, "limited virucidal" (effective against enveloped viruses, i.e. HBV, HIV, HCV), adenoviruses.

<b>Acting time:</b>	Hospitalism prophylactic incl. MRSA (VAH/DGHH, high contamination):	1 min.*
	tuberculosis prophylactic to VAH/DGHH (high contamination):	1 min.*
	Inactivation of cloaked viruses (e.g. HBV, HIV, HCV):	30 sec.

**Composition:** 100 g contain: 43,0 g Ethanol, 8,0 g 1-Propanol, 0,04 g Didecylmethylammoniumchloride, water, auxiliary materials.

**Note: H226** Flammable liquid and vapour. Flame point to DIN 51755: approx. 25 °C. Keep out of reach of children. Do not ingest! Application max. 50 ml per m<sup>2</sup>. The total quantity applied per room must not exceed 100 ml. Do not spray onto hot parts, live electrical components or into naked flames. Keep away from sources of ignition – do not smoke. Keep container tightly sealed. In case of contact with the eyes, rinse out thoroughly with water, and consult a doctor if necessary. \*Listed in the 5-min. column of the VAH/DGHH. Dispose of packaging for recycling only when completely empty. Dispose of remains under EAK 07 06 99.

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**Type of application: According to MPG:** ready-to-use quick disinfection agent for medical and dental use for the disinfection of non-invasive medical devices.

**According to the Biocide Directive:** ready-to-use quick disinfection agent for fast disinfection of small surfaces.

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**Acting time:** Hospitalism prophylactic incl. MRSA (VAH/DGHM, high contamination): 1 min.\*  
tuberculosis prophylactic to VAH/DGHM (high contamination): 1 min.\*  
Inactivation of cloaked viruses (e.g. HBV, HIV, HCV): 30 sec.

**Composition:** 100 g contain: 43.0 g Ethanol, 8.0 g 1-Propanol, 0.04 g Didecylmethylammoniumchloride, water, auxiliary materials.

**Note: H226** Flammable liquid and vapour. Flame point to DIN 51755: approx. 25 °C. Keep out of reach of children. Do not ingest! Application max. 50 ml per m<sup>2</sup>. The total quantity applied per room must not exceed 100 ml. Do not spray onto hot parts, live electrical components or into naked faces. Keep away from sources of ignition – do not smoke. Keep container tightly sealed. In case of contact with the eyes, rinse out thoroughly with water, and consult a doctor if necessary. \*Listed in the 5-min. column of the VAH/DGHM. Dispose of packaging for recycling only when completely empty. Dispose of remains under EAK 07 06 99.

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