INSTRUMENTENBAD PLUS

REF 200 724 Inhalt/Content/Contenu/Zawartość/Obsah/Tartalom: **e 10 L**

(GB) INSTRUMENT BATH PLUS

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(H) INSTRUMENTENBAD PLUS

Gefahr/Danger/Niebezpieczeństw/Nebezpečí/Veszély

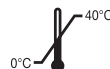
LOT

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0124

Stand: 220518

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☐ INSTRUMENT BATH PLUS

Type of application: Preparation for the manual cleaning and prophylactic disinfection of thermostable and thermolabile instruments in an immersion bath process. Not suitable for the final disinfection of semi-critical and critical medical devices!

Notes on application: Dilute the concentrate with 10°C to 30°C warm water (at least drinking water quality) (use a dosing bottle or dosing system) and place the instruments in the prepared stock solution immediately after use in an opened state. All surfaces and cavities to be disinfected must be completely wetted by the stock solution. Depending on the degree of soiling, the stock solution must be renewed (RKI recommendation: at least every working day). After the disinfection time, rinse the instruments thoroughly with drinking water (demineralised recommended) and dry. Do not mix with cleaning agents or other disinfectants! The reprocessing recommendations of the instrument manufacturer must be observed. Semi-critical medical devices must either be processed with a virucidally effective disinfectant or sterilised for final disinfection. Critical medical devices must be sterilised at the end. When using new materials, compatibility must be checked in advance. Suitable for use in an ultrasonic bath. The stock solution can be disposed of in the waste water. Dosing quantities and exposure times should be adhered to as precisely as possible and should not be lower.

Exposure times:

Bacteria according to EN 13727, EN 14561, VAH with high load:	1,5% 15 mins 1,0% 60 mins
---	------------------------------

Candida albicans according to EN 13624, EN 14562 VAH with high load:	1,5% 15 mins 1,0% 60 mins
Enveloped viruses (e.g. HBV, HIV, HCV) according to DVV/RKI with high load:	1,5% 15 mins 1,0% 60 mins
TbB (M. terrae) according to EN 14348, EN 14563, low load:	4,0% 15 mins 2,0% 60 mins

Safety notes: **H302** Harmful if swallowed. **H314** Causes severe skin burns and eye damage. **H400** Very toxic to aquatic life. Avoid release to the environment. Wear protective gloves/eye protection. IF ON SKIN (or hair): Take off immediately all contaminated clothing. Rinse skin with water slowly. IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing. Immediately call a POISON CENTER/doctor. Disposal of container in accordance with local / regional / national / international regulations. Contains didecylmethylammonium chloride, N-(3-aminopropyl)-N-dodecylpropane-1,3-diamine.

Directive 648/2004/EC: Contains 5-15% non-ionic surfactants, <5% cationic surfactants, <5% EDTA salts, fragrances.

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