Flächendesinfektion AF

Stand:080216

(GB) **Surface disinfectant AF**

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GB Surface disinfectant AF

Contents: e 5 ltr. [REF] 000 782

Strong cleaning concentrate, bio-degradable, aldehyde- and phenol-free For disinfection of medical products

Type of application: According to MPG: surface disinfection concentrate for disinfection of non-invasive medical devices. **According to biocide directive:** surface disinfection concentrate for disinfection of surfaces.

Notes on application: Prior to use the disinfection concentrate is diluted with tap water. **Do not mix with cleaning agents!** Before using on large surfaces with sensitive materials, for example, artificial leather, test first for material compatibility on a small hidden section. To avoid streaking, the treated surfaces should be wiped at regular intervals with a moist (water) disposable cloth.

Spectrum of activity: Bactericidal (incl. MRSA) acc. to VAH/DGfM, EN 1040, EN 1276; EN 13697; levurocidal (Candida albicans); "limited virucidal" (effective against enveloped viruses, i.e. HBV, HIV, HCV), and effective against rotaviruses and noroviruses.

Exposure times: Bacteria and Candida albicans (tested according to the standard methods of the DGfM, Status: 01.09.2001) under high mechanical load, expertise by Prof. Dr. Schubert:

5,0% 5 min./2,0% 15 min./1,0% 60 min./0,75% 240 min.

Bacteria according to DVG: 1,0/1,5% 30 min. (low/high load, 20°C)

"Limited virucidal" (enveloped viruses, i.e. HBV, HIV, HCV) acc. to RKI/DW: 3,0% 15 min. (with load) / 2,0% 15 min. (without load)

Rotaviruses according to BGfV/DV: 1,0% 1 min.

Noroviruses according to EN 14478: 6,0% 30 min. (low load)

Bacteria and Candida albicans: Entered in VAH / DGfM List: 2,0% 15 min./1,0% 60 min.

100 g contain: 3,2g N-Alkyl(C12-C18)-N-benzyl-N,N-dimethyl ammonium chloride, 3,2g N,N-Didecyl-N,N-dimethyl ammonium chloride, 3,2g N-Alkyl(C12-C19)-N-ethyl benzyl-N,N-dimethyl ammonium chloride

Safety notes: H315 causes skin irritation. H318 Causes serious eye damage. H319 causes severe eye irritation. H400 Very toxic to aquatic life. Wear protective gloves / eye protection. Avoid release to the environment. **IF IN EYES:** Rinse carefully with water for several minutes. Remove contact lenses if possible. Continue rinsing. In case of skin irritation: get medical advice/attention. If eye irritation persists: Get medical advice/attention. **IF IN CONTACT WITH SKIN:** wash with copious amounts of water and soap. Disposal of contents / container in accordance with local / regional / national / international regulations.

Only pass completely emptied containers to recycling. Dispose of residual material acc. to EAK 07 06 99.

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