# Bohrerbad

Stand 1523

GB **Drill bath**

VAH/DGHM-  
gelistet

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PL **Kąpiel do wiertel**

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## GB Drill bath

Contents: e 10 ltr. ready-to-use solution REF 000 731

**Type of application:** For cleaning and disinfecting dental drills and rotary instruments of all kinds.

**Notes on application:** The drill bath is a ready-to-use solution which can be used instantly. Place instruments into the undiluted solution immediately after use. Drills must be fully wetted by the disinfectant. Remove following the exposure time, rinse with water and dry. Do not mix with cleaning agents or other disinfectants. Not suitable for aluminium.

**Spectrum of activity:** Bactericidal according to VAH/DGHM, EN 13727 and EN 14561. Levurocidal (Candida albicans) according to VAH/DGHM, EN 13624 and EN 14562.

Tuberculocidal according to VAH/DGHM, „limited virucidal“ (effective against enveloped viruses, e.g. HBV, HIV, HCV) according to BGA(RKI)/DVV.

<b>Exposure times:</b> Bacteria (tested according to the standard methods of the DGHM, as of 01/09/2001) and EN 13727, EN 14561 (high load): Candida albicans (tested according to the standard methods of the DGHM, as of 01/09/2001) and EN 13624, EN 14562 (high load): TbB (tuberculosis) inactivation (M. terrae, tested according to the standard methods of the DGHM, as of 01.09.2001): „Limited virucidal“ (enveloped viruses, i.e. HBV, HIV, HCV) according to BGA(RKI)/DVV (with load):	conc.: 15 min. conc.: 15 min. conc.: 30 min. conc.: 15 min.
---	--

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