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(GB)

INTENDED PURPOSE

This medical device is used in conjunction with Dental air polishers. This medical device is intended for prophylactic treatments of dental and prosthetic surfaces.

1 – DESCRIPTION

M+W Prophy Powder has been developed exclusively for prophylactic supragingival treatment. The powder can be used with all standard air polishers. 250 g multidose bottle, with pourer

2 – INDICATIONS

• Removal of dental plaque (orthodontic treatments or before whitening), spot removal. Removal of provisional dental cement residues. • Polishing teeth after calculus removal. • Interproximal cleaning, cleaning fissures and sulci. • Cleaning bands and brackets in the mouth. • Cleaning and preparation of the tooth surface before etching.

3 – PROPERTIES

The grinding properties of M+W Prophy Powder do not damage tooth enamel or gums. M+W Prophy Powder: no pain or excessive bleeding, perfect for supragingival maintenance treatment. With its differently emphasised tastes it provides a sensation of freshness on the mouth.

4 – COMPOSITION

M+W Prophy Powder: sodium bicarbonate, hydrophobic silicic acid, sodium saccharine, excipients, natural essential oils or aromas.

5 – PRECAUTIONS FOR USE

Before using the powder check the general health of your patient. • Observe the terms and conditions of use specified by the manufacturer of the air polisher. • Patient and doctor should wear appropriate protection (glasses, masks). • Do not direct the jet directly at the gum. • Patient should rinse out the mouth with an antiseptic solution before and after treatment. • Apply Vaseline(R) to the patient's lips before polishing. • Hold a large suction tube near the treatment site (to reduce contamination by airborne bacteria). • Rinse with copious amounts of water after contact with skin or eyes. • Do not swallow.

6 – CONTRAINDICATIONS

• Known allergy to one of the constituents. • Patients who suffer from endocarditis. • Patients with immune deficiencies. • Patients under treatment with antibiotics, chemotherapy or radiotherapy. • Patients on a low-salt diet. • Patients with diabetes, asthma or respiratory problems. • Haemophiliacs. • Pregnant or nursing women.

7 – INSTRUCTIONS FOR USE

Observe the instructions for use of your air polisher. General: • Keep the “jet to surface” angle at 45° to 60°. • Keep the “spray to surface” distance at approx. 6 mm. • Make small circular movements on the tooth. • Direct the jet towards the occlusal margin.

8 – STORAGE

Do not use after the expiration date on the outside of the package. Close the bottle tightly after use (use up quickly after opening). Store at room temperature (< 25°C) away from moisture and light.

9 – DISPOSAL OF UNUSED POWDER

Do not dispose of in drains or rivers. Do not dispose of with domestic waste. **Note:** Federal law of the United States of America restricts the use of the device to qualified dental personnel or to persons under their supervision.

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