D Provisorischer Füllungszement

Definition

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GB Temporary filling cement

Definition

Zinc oxide and syntetic materials temporary filling cement, self hardening, ready to use.

Contra-indications

None.

Side Effects

In exceptional cases an allergic reaction can occur, refrain from using this product in those cases.

Advantages

- Thixotropic : soft before setting to allowed a perfect obturation
- No pulpar or gum irritations because of its neutral PH
- No expansion when setting, due to the presence of a high quality syntetic materials
- Fast setting in presence of saliva
- Contains no eugenol : does not hinder polymerization of resins and composites
- Contains fluoride : cariostatic action

Jar form

Remove the desired amount of cement with a spatula Tightly close the jar after use.

Application

Apply the cement, hardening start immediatly.

Presentation

Three consistencies

White / Normal, Pink / Hard, Grey / Soft

Storage & shelf-life

Do not stock above 25°C, 4 years shelf-life.

Contraindications

The use of Proviso is contraindicated if the patient is known to be allergic to any of the ingredients in Proviso.

Important notice

- Do not throw away the box and the instructions for use until the product is completely finished.
- High humidity could damage the packung on which important instructions and legal information are printed.
- Storage at a too high temperature could cause to age prematurely and inhibit its polymerisation.
- Storage at a too low temperature leads to the decomposition of the product which will ender it completely unusable.

These materials have been developed solely for use in dentistry. Processing should be carried out strictly according to the Instructions for Use. Liability cannot be accepted for damages resulting from failure to observe the Instructions or the stipulated area of application. The user is responsible for testing the material for its suitability and use for any purpose not explicitly stated in the Instructions. Descriptions and data constitute no warranty of attributes and are not binding.

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