Septoprint Konzentrat

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VAH/DGHM-listed

Content: e 1 ltr. REF 100 725

Type of application: The impression disinfectant is used for disinfecting and cleaning all dental impression materials, as well as prosthetic work.

Information on usage: The concentrate must be diluted with water before use. Place the Impression according to the below-mentioned contact time in to the diluted prepared solution. Take care that the impression is completely covered with the solution. In case of alginates and hydrocolloids, it is recommended to briefly immerse the impression in the solution and subsequently dry for 5 minutes. Do not mix with cleaners or other disinfectants.

Spectrum of action: Bactericidal (incl. MRSA) according VAH/DGHM, EN 1040, EN 1276, EN 13697; Levurocid (Candida albicans) "virucidal with limited effect"

tiveness" (effective according enveloped virus , e. g. HBV, HIV, HCV) and effective against rotavirus.

Contact time:

Bactericidal and Candida albicans (according standard method of DGHM, status 01.09.2001) as surface disinfection under high stress):

5% 5 min.
2% 15 min.

Bactericidal according EN 1040, EN 1276, EN 13697 (high stress).
1% 15 min.

Candida albicans according EN 1275, EN 1650, EN 13697 (high stress):
1% 15 min.

Virucidal with limited effectiveness (enveloped virus e. g. HBV, HIV, HCV) according RKI/DVV:

5% 5 min. (under stress)
1% 15 min. (without stress)

Rotavirus according BGVV/DVV:
1% 15 min

100 g contains: 3.2 g dialkyl dimethyl ammonium chloride, 6.4 g alkyl (aryl) - dimethyl ammonium chloride

Warning: H315 Causes skin irritation. **H319** Causes serious eye irritation. **H400** Very toxic to aquatic life. Wear protective gloves / eye protection. **IF IN EYES:** Rinse carefully with water for several minutes. Remove contact lenses if possible. Continue rinsing. Call the **POISON INFORMATION CENTRE** or a doctor immediately. **IF SWALLOWED:** rinse mouth. **DO NOT** induce vomiting. **IF IN CONTACT WITH SKIN:** wash with copious amounts of water and soap. Store in a closed container. Disposal of contents / container in accordance with local / regional / national/international regulations. Contains iron phosphoric acid.

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